

Premium Fat-Transfer and Body-Contouring



INNOVATIVE

novel drive system which can be adjusted to the intensity and effectiveness of the water-jet to the user needs

EFFECTIVELY

easy fat transfer without the need for time consuming processing

VERSATILE

The modern and standardized concept of operation adjust settings to the individual needs of the surgeon.

●● Your stake in the future


**of aesthetic, reconstructive
and regenerative medicine**

The innovative body-jet® evo represents the 2nd generation of Water-jet Assisted Liposuction devices (WAL). With the help of a fan-shaped, pulsating water-jet, the fat can be gently separated away from the surrounding tissue and evacuated at the same time.

Thanks to a novel drive system for the body-jet® evo, the intensity and effectiveness of the water-jet can be adjusted to the user needs, ensuring a virtually pain-free treatment. The surrounding connective tissue, nerves and vascular structures remain undamaged resulting in higher patient safety, faster healing and less pain and bruising. Moreover, the gentle effect of the water jet allows an easy and effective fat transfer even in patients with higher fat volumes without the need for time consuming processing which may lead to cell damage.

The body-jet® evo comprises a modern and standardized concept of operation with various preset parameters for liposuction and fat transfer as well as the option to adjust settings to the individual needs of the surgeon.



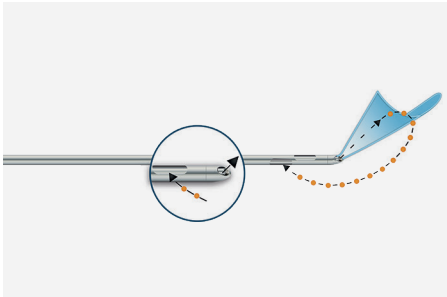


Less need for time
Two in one

aspiration and infiltration at the same time




Operation
Easy to operate via touch screen

- easy and intuitive handling
- upgradable platform for further applications
- integrated help function

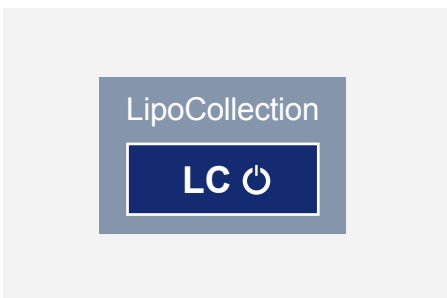

Quality
Ensuring highest fat quality

- for reliable harvesting of viable fat cells
- for lipofilling

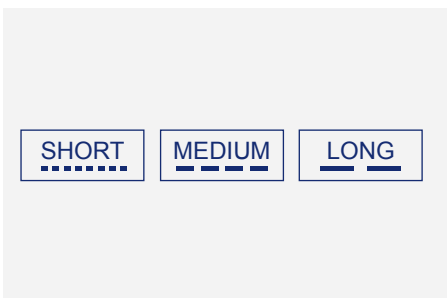

Efficiency
Adjustable pulse effects

- fastest surgery time
- less pain for the patient
- efficient workflow


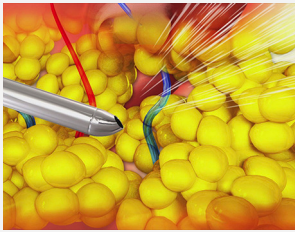
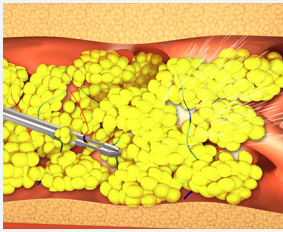
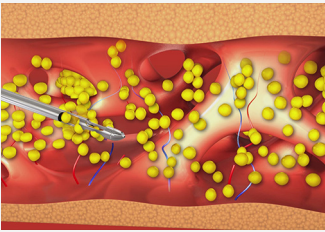


Precision
Flexible for liposuction and fat transfer


- standard system: for fast and precise fat collecting
- Biofill system: cost effective for harvesting small fat volumes

Phases of WAL

●● Waterjet-Assisted Liposuction


Phase 1		Phase 2	Result
Infiltration of the tissue, using gentle water-jet		Simultaneous irrigation and aspiration	
			
No tumescent technique		The fan-shaped waterjet flushes out small clusters of fat tissue. The water-jet moves easily through the toughest tissues. No need for aggressive cannula movement.	Low collateral damage to surrounding tissues

●● Pulse effects

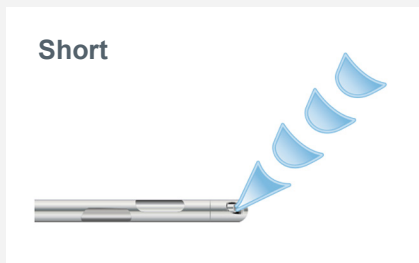


short and quick pulse

- most effective pulsation for guiding the cannula through fibrotic tissue
- easy to learn and to perform for skilled and unskilled users

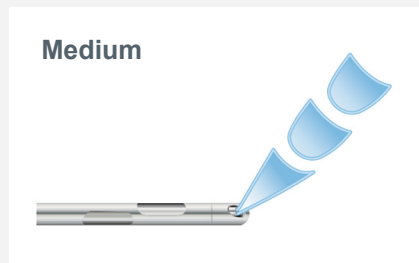

basic pulsation mode

- most successful water-jet of the 1st body-jet® generation
- scientifically proven clinical results

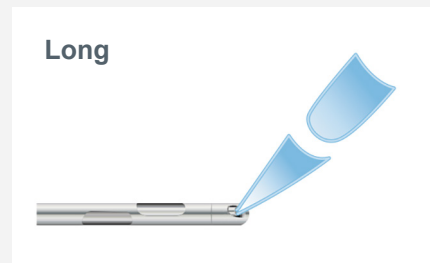

long pulsation mode


- fast, time-saving infiltration of large volume of solution
- designed for quick tissue expansion

● body-jet® evo

Our product

Efficient. Versatile. Intuitive.

The premium class of water-jet assisted fat-transfer and body-contouring

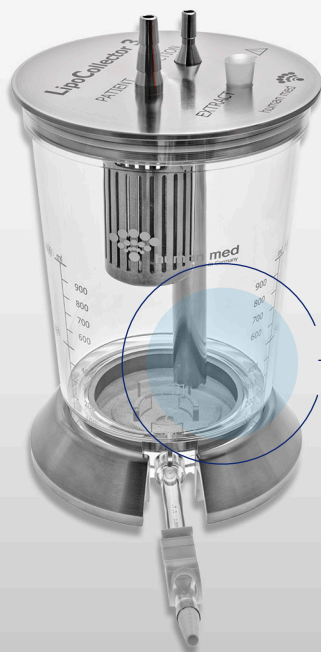

Fat management

Fat collection

- 1st phase:** Fast and gentle harvesting without tumescent technique
- 2nd phase:** Direct fat collection using LipoCollector® or FillerCollector, no centrifugation or processing required
- 3rd phase:** Extraction and direct re-injection of the fat into the recipient areas



●● Leader in water-jet technology

Human Med AG is an innovative medical technology company in the field of water-jet surgery with its headquarters in Schwerin / Germany. With its unique technology for hydrodynamic separation and harvesting of tissue and cells, particularly from the subcutaneous fat tissue, the company is leading the international market of plastic-reconstructive and aesthetic surgery. Human Med AG can build on long-term experiences with water-jet assisted systems allowing precise control and selective tissue treatment. On this basis, the company developed the tissue-sparing method of Water-jet Assisted Liposuction and Lipotransfer (WAL) that is used in the aesthetic market, as well as plastic and reconstructive surgery for body-contouring, breast augmentation and reconstruction, and for treatment of soft tissue defects, scars and chronic wounds.

●● Technical details

body-jet[®] evo

Dimensions: 54 (B) x 60 (T) x 178 (H) cm

Weight: 65 kg

Power supply: 230 VAC, 6 A, 50 Hz

Safety class according IEC 60601-1: I

OP applicator safety class: BF

Approval: CE 0482

Device class acc. to Council Directive

93/42/EEC: II a

body-jet[®] is a registered trademark owned by Human Med AG, Germany. body-jet[®] technology is protected by U.S., European and other patents.

Human Med AG
Wilhelm-Hennemann-Str. 9
19061 Schwerin
Germany

Tel.: +49 (0)385 395 70 0
Fax: +49 (0)385 395 70 29
info@humanmed.com
www.humanmed.com