

Indication	Type of Lens	Energy	Distance	i-stack	Scan Patten	Pass	Remark/Caution
Scars	300um	13 - 16mj	1mm	1	Random	4	Use Circle Shape, Max Size. Overlap 30 - 50% and repeat for 4 passes on lesion. Then 1 pass on the rest.
Light Rejuve	300um	8mJ	1mm	1	Random	2 - 4	Use Circle Shape, Max Size. Overlap 30 - 50% and repeat for 2 - 4 passes.
Rejuvenation / Wrinkles / Pore Reduction	300um	10 - 12mj	1mm	1	Random	2 - 4	Use Circle Shape, Max Size. Overlap 30 - 50% and repeat for 2 - 4 passes on Lesion. Then 1 pass on the rest.
Acne	300um	8 - 10mj	1mm	1	Random	4	Use Circle Shape, Max Size. Overlap 30 - 50% and repeat for 4 passes on Lesion. Then 1 pass on the rest.

